

## TrioMed Antimicrobial Self-Adhesive Absorbent Dressing

### INTENDED PURPOSE

The TrioMed Antimicrobial Self-Adhesive Absorbent Dressing is an individually packaged, single use, disposable dressing intended as a primary fixation dressing for a wide range of wounds or as a secondary dressing to cover or secure objects. The dressing is treated with a patented iodine-based antimicrobial to protect the external surface from microbial deterioration. Hypoallergenic.

### INSTRUCTIONS:

Product can be worn for up to 5 days, or as per instructions from your nurse/physician.

Product has a 5 year shelf life.

For external use only.

Single use. Once removed it must be discarded as it will not adhere appropriately to skin anymore.

Apply on clean, dry and oil or cream free skin.

Gently clean the wound and dry surrounding skin. Remove from package, peel away paper backing and place over intended area.

Store at room temperature away from direct sunlight.

Dispose of according to your country's regulations.

### CONTRAINDICATIONS

Discontinue use and consult a physician if irritation or sensitivity develops.

This product contains iodine, it should not be used by persons with known iodine hypersensitivity.

### WARNINGS:

The product helps protect against certain particulate contaminants but does not eliminate exposure to or risk of contracting the disease or infection.

Any serious incident that occurs when using this device should be reported to the manufacturer and the national competent authority.

### PRODUCT BENEFITS:

Non-cytotoxic, latex free and non-flammable

The outer active layer with TrioMed antimicrobial technology (iodine based) is to protect the external surface from microbial deterioration. The iodine is embedded in the external surface therefore does not leach or escape the material, its purpose is to render bacteria and viruses inactive when they come into contact.

	 	<b>Obelis S.A.</b> Bd. Général Wahis 53 1030 Brussels, Belgium Tel: +(32) 2 732-59-54 Fax: +(32) 2 732-60-03 mail@obelis.net	 	<b>I3 BioMedical Inc.</b> 14163 boul. Curé Labelle Suite 50 Mirabel, Québec, Canada J7J 1M3 WWW.I3BIOMEDICAL.COM
-------------------------------------------------------------------------------------	----------------------------------------------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------